

Getting Started with the Versah® Guided Surgery System

1.

Determine the Following:

- Tooth #
- Implant Diameter

Versah® Guided Surgery System Price Breakdown

Item	Unit Price
G-Stop™ Vertical Gauge ...	\$24 each
G-Stop™ Key	\$69 each
C-Guide™ Sleeve.....	\$19 each
G-Stop™ Holder	\$95

2.

Plan the Case:

- Take a CBCT Scan, Dicom File, and impressions of the site and the opposing jaw.
- Contact a Versah® Certified Lab to begin the process of printing a custom C-Guide™ Sleeve.

Versah Certified Labs

A full list of certified labs available at versah.com/guided-surgery-system/

Dorsey Dental

www.dorseydentallab.com

Cinzara

www.cinzara.com

ROE Dental Lab

www.roedentallab.com

3D Diagnostix

www.3ddx.com

D3 Solutions

www.d3implantsolutions.com

Pro-Art Dental Laboratory

www.pro-artdentallab.com

I Use CEREC

- Set up your software to produce the guide with a hole that matches the Versah® C-Guide™ Sleeve outer-diameter.
- Create buccal window to match the Versah® C-Guide™ Sleeve opening.
- Order the C-Guide™ Sleeve and the compatible G-Stop™ from Versah®.

I Print My Own Guide

- Set up your software to produce the guide with a hole that matches the Versah® C-Guide™ Sleeve outer-diameter.
- Create buccal window to match the Versah® C-Guide™ Sleeve opening.
- Order the C-Guide™ Sleeve and the compatible G-Stop™ from Versah®.

I'd Like to Use My Lab

- Versah® is committed to certify your dental lab to produce the Universal C-Guide™ Surgical Guide. Please call us at: 517-796-3932.



Versah® Guided Surgery System Compatible Software

A full list of compatible software available at versah.com/guided-surgery-system/



3shape.com



BlueSkyBio

blueskybio.com



coDiagnostiX™

codiagnostix.com

PLANMECA

planmeca.com



powered by swissmeda

mysmop.com

Sizing the Versah[®] Guided Surgery System

G-Stop™ Gauge




Available
Sizes

S

Placement up
to 3.25mm
Implant

Densah[®] Bur
Compatibility

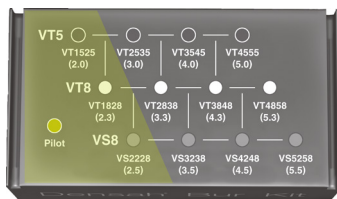


G-Stop™ Vertical
Gauge + Key



OD 4.2mm

C-Guide™

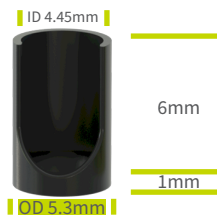


M

Placement up
to 4.3mm
Implant



OD 5mm

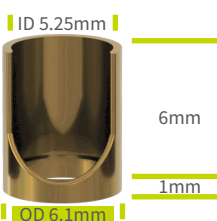

L

Placement up
to 5.3mm
Implant



OD 6mm




XL

Placement up
to 6.2mm
Implant



OD 7mm

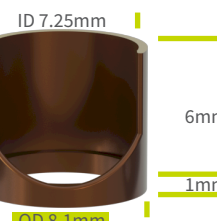

C-Guide™
Sleeve



Available G-Stop™ Drilling Depths



www.versah.com

Versah[®]
THE OSSEODENSIFICATION COMPANY

©2018 Huwais IP Holdings LLC. All rights reserved. Versah[®] and Densah[®] are registered trademarks of Huwais IP Holdings LLC. HT017 REV07